



1. On-Off
2. Confirm Menu and static mode
3. Max Inflate and LAL
4. Patient Comfort Control level (manual)
5. Alternating Pressure
6. Patient set up (height and weight)
7. Lock Out
8. Pulse
9. Silent
10. Cycle Time
11. Bolster...optional
12. Automatic weight adjustment

For Auto-Set mode

1. Inflate completely using Max Firm mode.
2. Select Auto-Set mode.
3. Activate Alternating Pressure mode.

If required, to change the deflated cell %, hold down the AP button for 5 seconds and adjust the deflation % using the +/- arrows.

*A cycle change is required for automatic pressure adjustment to work.

K2 installation

- a) Attach the mattress to the moving part (not the base) of the bed using the appropriate cords.
- b) Plug in the power cord.
- c) CPR key (CPR) closed position on the left foot.
- d) Connect the tubes to the pump.
- e) Switch on, ON/OFF button.
- f) Wait for inflation (approx. 5-10 minutes).
- g) Install bedding

***** Make sure the CPR valve is in the Closed position before inflating the mattress *****

TROUBLE SHOOTING GUIDE

⚠ THE FOLLOWING INFORMATION IS INTENDED ONLY FOR FACTORY-AUTHORIZED SERVICE FACILITIES AND FACTORY-QUALIFIED SERVICE PERSONNEL.

Leika Medical Equipment can provide technical assistance to qualified technical personnel. Contact **Leika Medical Equipment** Customer Service for more information at 450-659-1991.

PROBLEM	CAUSE	THE SOLUTION
A. Mattress does not inflate properly	<ol style="list-style-type: none"> 1. Mattress hose disconnected 2. Bent or split air hose 3. Major leak in air cushion or cover cushion 4. Folded or split manifold 5. Control unit not working 6. Compressor malfunction 	<ol style="list-style-type: none"> 1. Connect hose fittings and lock in place 2. Unfold the hose or replace the split hose 3. Replace leaking air cushion or cover cushion 4. Unfold manifold or replace split manifold 5. Return control unit to factory for repair 6. Return control unit to factory for repair
B. No power supply	<ol style="list-style-type: none"> 1. Control unit off 2. Power cord unplugged 3. No power in power the source 4. Power failure 5. Blown fuse 	<ol style="list-style-type: none"> 1. Check the power source and switch on the device 2. Connect cord to power source 3. Check that the power source is supplied and switch it on 4. Wait for the power source to be supplied 5. Return control unit to factory for repair
C. Control unit does not respond	<ol style="list-style-type: none"> 1. unit locks 	<ol style="list-style-type: none"> 1. Unplug the power cord from the control unit, wait 15 minutes and plug the cord back in to reset the control unit.

Maintenance:

Regular disinfection according to your protocol (no phenol).

CARDIOPULMONARY RESCUSITATION

Turn the red CPR valve on the left-hand side of the foot mattress.

T. 450. 659.1991 T. 877. 659.1991 F. 450. 659.4925

9585, rue Ignace, local D, Brossard Qc J4Y 2P3

******* Make sure the CPR valve is in the Closed position before inflating the mattress ******