

Domus 1

1. Tuck the extension flaps under the standard mattress to ensure stabilization.
2. Plug in the power cord (bottom of the pump).
3. Connect the mattress air hoses to the pump.
4. Turn the unit on by pressing the ON/OFF (I/O) switch (left side of the pump).
5. Adjust the patient's weight using the adjustment dial.
6. Install the bedding.

Facade panel



1- PRESSURE SETTINGS:

- To adjust the pressure according to the patient's weight (lbs), select the patient's weight using the adjustment dial on the front of the pump.

1- ON/OFF:

This button allows you to turn the pump on and off.

2- TRANSPORT MODE or POWER FAILURE:

Do not remove the air hoses from the pump.

3. TROUBLESHOOTING GUIDE

Situation	Possible Cause	Solution
Perte de pression	1.Connector improperly connected or tubes jammed 2.Leak in the system	Check: <ul style="list-style-type: none">• Ensure that all pipe fittings along the mattress are properly in place.• Contact technical support
Problème d'alimentation	1.A power outage has occurred. 2.The power cord is unplugged.	Check: <ul style="list-style-type: none">• Press the Alarm Mute button

4- MAINTENANCE

Regular disinfection according to your protocol (no phenol)