


Pro-care Optima

1. Unroll the mattress onto the bed and securely fasten it to the movable section (not the base) using the appropriate straps.
2. Hang the control unit (pump) at the foot of the bed.
3. Plug the power cord into a grounded wall outlet.
4. Ensure the hose connector is properly inserted into the pump. Press firmly until you hear a click.
5. Make sure the CPR valve is closed before inflation (located at the foot of the mattress).
6. Turn the pump on using the Power (ON/OFF) button.
7. Allow approximately 5 to 10 minutes for the mattress to reach optimal pressure.
8. Once the pressure is established, install the bedding and position the patient.

Facade panel

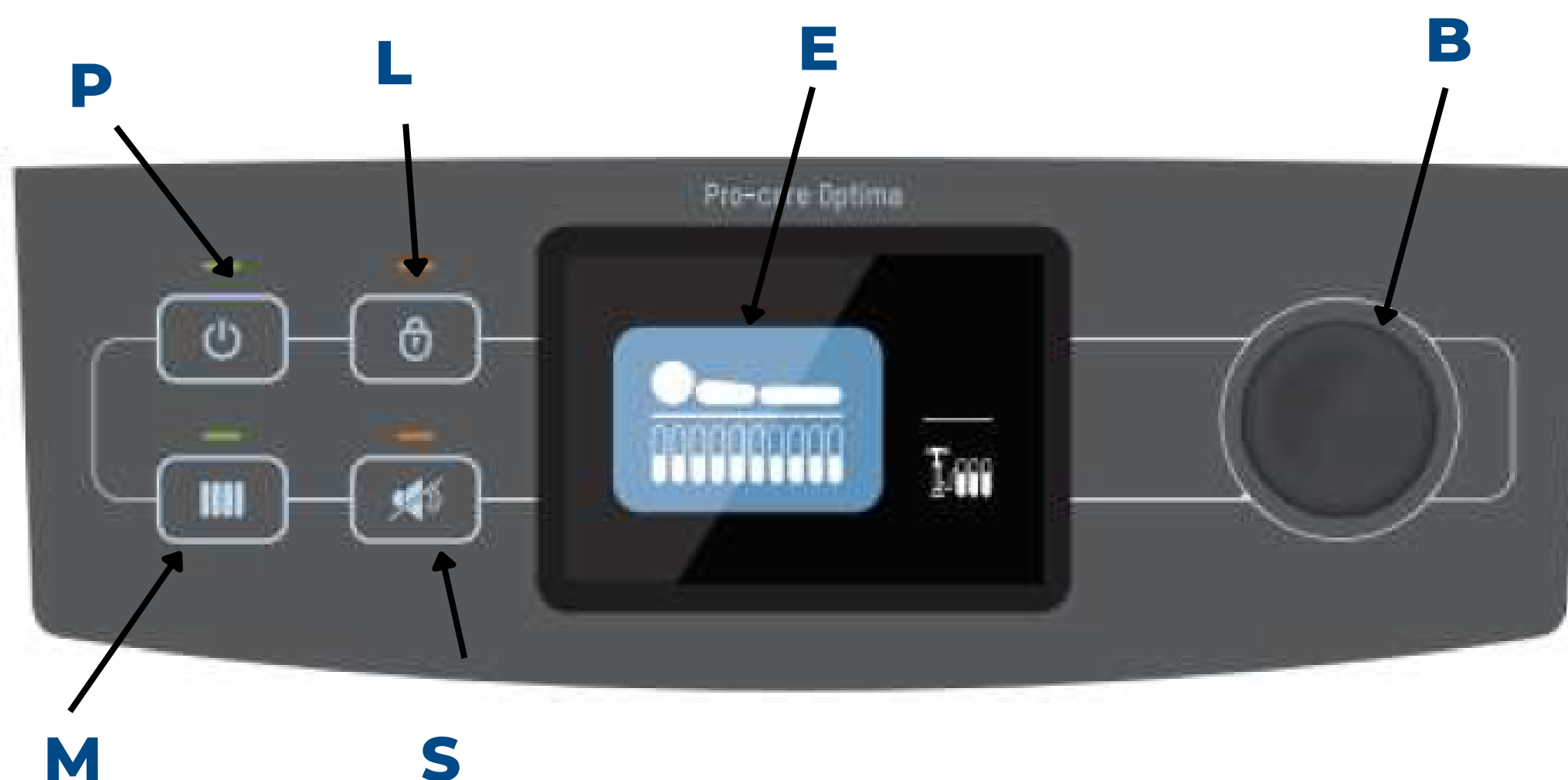

1- CONTROL PANEL ADJUSTMENT

P	Power key to turn on and turn off pump
L	Panel lock key to lock or unlock screen
E	Écran permettant d'accéder aux menus ou de visualiser les messages d'erreur
B	Knob is to choose and confirm settings displayed on the LCD
5	Silent/Silence
M	Max firm key is to reach maximum operating pressure

2- CARDIAC RESUSCITATION

- Turn the red CPR valve located on the left side at the foot of the mattress.
- The mattress will deflate rapidly to allow for resuscitation.

3- TRANSPORT MODE OR POWER FAILURE

An integrated safety foam supports the patient during transport or in the event of a power failure. No additional action is required.

4- TROUBLESHOOTING GUIDE

Situation	Possible cause	Solution
The mattress does not inflate properly.	<ul style="list-style-type: none">• Mattress hose disconnected• Air hose kinked or cracked• Major leak in the air cushion or the cover cushion• Manifold kinked or cracked• Control unit not functioning• Compressor malfunction	<ul style="list-style-type: none">• Connect the hose fittings and lock them into place.• Unkink the hose or replace the cracked hose.• Replace the leaking air cushion or cover cushion.• Unkink the manifold or replace the split manifold.• Return the control unit to the factory for repair.• Return the control unit to the factory for repair.
No power	<ul style="list-style-type: none">• Control unit turned off• Power cord unplugged• No power at the power source• Power outage• Blown fuse	<ul style="list-style-type: none">• Check the power source and turn the unit on.• Connect the power cord to the power source.• Ensure the power source is functioning, then turn the unit on.• Wait until power is restored.• Return the control unit to the factory for repair.
The control unit does not respond.	The unit is locked.	<ul style="list-style-type: none">• Unplug the power cord from the control unit, wait 15 minutes, then plug the cord back in to reset the control unit.

5- TROUBLESHOOTING GUIDE (error codes)

Code No.	Diagram	Description
01 · 02 · 05		Low pressure alarm: body section
03 · 04 · 06		Low pressure alarm: foot section
07		Low pressure alarm: head section
19		Position sensor is not connected
20		Quick connector is not connected
21		Low battery volume (< 25 %)
21		Low battery volume (< 10 %)
24		Connect a wrong bed

5. MAINTENANCE:

Regular disinfection according to your protocol (no phenol).