


## Serene Elite


1. Secure the mattress to the movable part (not the base) of the bed using the appropriate cords.
2. Connect the power cord (right side of the pump).
3. Connect the quick-connect fitting of the mattress air hose to the pump;
  - Ensure that the connector is in the correct position.
  - Connect from bottom to top, you will hear a "click".
4. To turn on, press the O/I switch (right side of the pump) and press the button on the control panel to start the pump.
5. Max Firm mode runs automatically for the period of swelling (approximately 20 minutes).
6. Set up the bedding.

### Facade panel

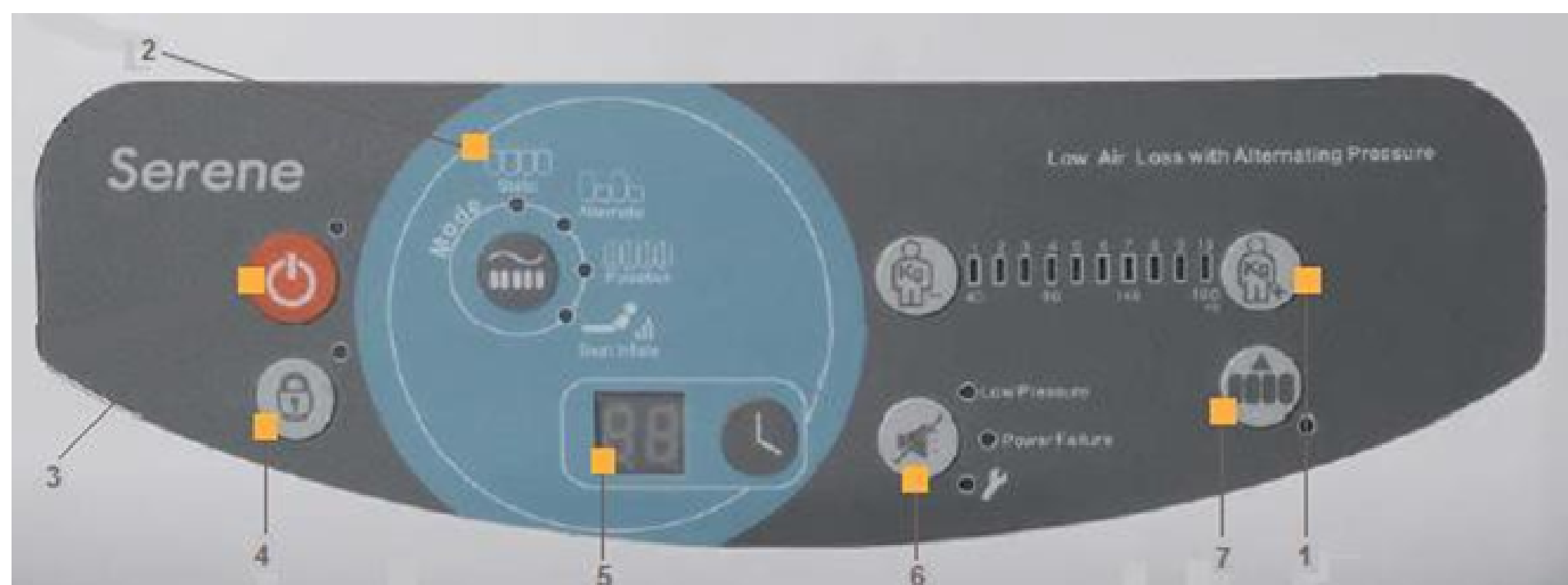

### 1- PRESSURE SETTINGS:

- To adjust the pressure according to the patient's weight (lbs), press the button to increase the level on the scale from 90 to 1000 lbs.

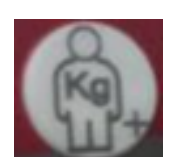

- To decrease the firmness of the cells, press the button



## 2- THERAPY:

**Static: Static mode offers continuous low contact pressure.**

- **Alternate: The alternating mode allows for continuous modification of pressure points, for optimal pressure relief.**
- **Seat Inflate: This mode provides additional support when the patient needs to remain seated. It can be combined with the alternating or static mode.**

## 3- ON/OFF:

This button allows you to turn the pump on/off.

NOTE: The on/off switch on the side of the pump must be in operation.

## 4- LOCK

This locking mechanism prevents accidental changes to the pump settings. When activated, the green indicator light illuminates. To activate or deactivate this mechanism, press this button for 3 seconds. This mechanism automatically reactivates 5 minutes after being deactivated.

## 5- CYCLE TIME:

Four cycles are available: 5, 10, 15 and 20 minutes. By pressing the "cycle time" button, you can select one of the four cycles.

## 6- ALARM MUTE:

Audible alarm silence button. When activated, it temporarily suspends the alarm; if the situation is not resolved within 3 minutes, the alarm will reactivate.

- **Low Pressure: The Low Pressure indicator light illuminates when the pump detects insufficient pressure inside the mattress. This condition also triggers an audible alarm. This occurs when the pressure is insufficient to provide the necessary support for the patient. The light turns off when the pressure is restored.**

- **Power Failure:** In the event of a power outage or when the power cord is unplugged and the appliance switch is closed, the indicator light will illuminate and an audible alarm will sound. Press the alarm deactivation button.
- **Service:** The indicator light illuminates when there is an internal pump failure. Contact technical support.

### **7- MAX FIRM:**

This mode activates automatically for approximately 20 minutes when the pump is switched on, allowing the cells to be inflated to their maximum pressure. Once inflated, they form a firm and stable surface, facilitating treatment protocols and/or patient mobilization.

### **8- CARDIAC RESUSCITATION**

To perform CPR, press the Quick Connector release button to quickly detach the connector from the pump and evacuate the air.

NOTE: There are 2 CPR buttons on bariatric mattresses

### **9- TRANSPORT MODE or POWER FAILURE**

Do not remove the Quick Connector from the pump.

## 10. TROUBLESHOOTING GUIDE

<b>Situation</b>	<b>Cause Possible</b>	<b>Solution</b>
<b>Low pressure (indicator light and alarm)</b>	<ol style="list-style-type: none"><li>1.Connector improperly connected or tubes jammed</li><li>2.Leak in the system</li></ol>	Check: <ul style="list-style-type: none"><li>• Ensure that all pipe fittings along the mattress are properly in place.</li><li>• Contact technical support</li></ul>
<b>Power failure</b>	<ol style="list-style-type: none"><li>1.A power outage has occurred.</li><li>2.The power cord is unplugged.</li></ol>	Check: <ul style="list-style-type: none"><li>• Press the Alarm Mute button</li></ul>
<b>Service (indicator light)</b>	Internal pump failure	<ul style="list-style-type: none"><li>• Contact technical support</li></ul>

## 11. MAINTENANCE:

Regular disinfection according to your protocol (no phenol).